

Blood sampling and shipping instructions

1. Ask your veterinarian to take a blood sample of your dog.
2. Collect 0.5 - 1 ml of blood to a purple-capped **EDTA-tube**. Invert the tube 8-10 times to prevent clotting.
3. Clearly label the tube with the dog's name. Make sure that the sample is packed properly, for example, you can place blood tube into small plastic pack to prevent damage or leakage.
4. Please, fill out the request order form and send it together with the labeled tube with blood to the laboratory.



Shipping:

Please, use professional courier for shipping (DPD, UPS or others). Note, that if you send a parcel by standard international post, the parcel might be significantly delayed or even lost. If asked by a courier, define the content as "exempt animal specimen - samples".

Shipping address:

GenoCan Ltd
Krizikova 70
612 00 Brno
Czech Republic